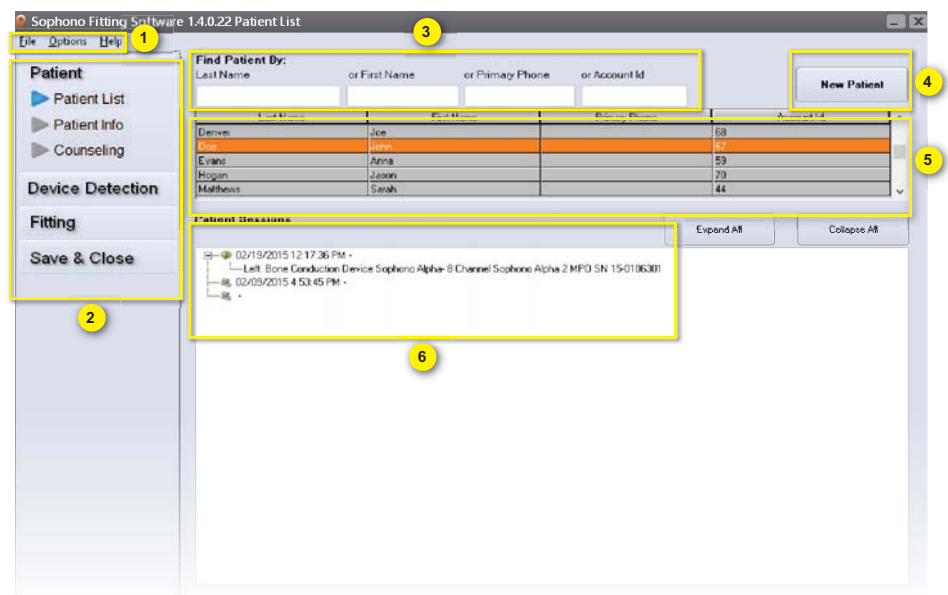
SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Use this screen-by-screen guide to find detailed explanations for the most important aspects of the Fitting Software.

Main menu > Patient > Patient List (main screen)

1. File: Import/Export patient data; Save; Close
Options: Show/Hide Hints; select programmer
Help: Help Files
2. Main menu – easy access to all screens
3. Patient search fields
4. New Patient button
5. Alphabetical patient list
6. Selected patient's stored sessions

SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Main menu > Patient > Patient Info

Enter in patient's contact information and audiogram. Be sure to enter in patient's hearing loss type and bone conduction thresholds to help with prescriptive fitting options. Select binaural fitting if appropriate. Clear or copy patient's audiogram as needed.

1. Enter in patient's information (only last name is required).
2. Audiograms, BC and AC required for prescriptive fitting
 - BC: bone conduction
 - AC: air conduction
3. Hearing loss types; required for prescriptive fitting options. Choose Binaural if appropriate.

Main menu > Counseling

Helps explain the device to potential patient. Contains links to online videos, processor information, and audiogram information.

1. Links to explanatory online videos
2. Display of processor colors
3. Detailed processor description
4. Patient's audiogram. None selected shows speech banana.

SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Main menu > Device Detection

Confirm whether or not a device is detected by the software. Select Autofit options or manual adjustment of the device.

1. Device detection status displays what device, if any, has been detected.
2. Choose to use the connected device or enter into simulation mode.
3. Select Quick Fit or Manual Adjustment
4. Quick Fit options
5. See Autofit Options below
6. Select Fitting after all selections have been made.
7. Select Retry if device was not detected but is connected.

The screenshot shows the 'Device Detection' window. On the left, a box displays 'Sophono Alpha 2 MPO' and 'Sophono Alpha- 8 Channel - Bone Conduction Device'. Below this are radio buttons for 'Use Connected Device - SN: 15-0106301' (selected) and 'Simulate'. A 'Programming Choice' section has 'Quick Fit' selected over 'Manual Adjustment'. The 'Patient Type' section has 'Adult' selected over 'Pediatric under 12', with a 'DAI' checkbox. Below are 'Active Memories' (1-4) and 'Autofit Options' (Sophono Ideal Fit, Noise Fit). A device image is shown at the bottom. On the right, a 'No Device Detected - Left' box contains instructions and 'None'/'Simulate' options. A red 'X' over a device image is also present. At the bottom are 'Fitting' (6), 'Retry' (7), and 'Cancel' buttons.

Autofit Options

Sophono Ideal Fit: Maximum gain with preset notch filters to reduce feedback. Best used for patients with SSD or mixed hearing loss who need ample gain.

Everyday fit: Prescriptive Omni Plus fitting that uses the patients bone conduction thresholds and type of hearing loss.

Noise fit: Prescriptive Adaptive Directional fitting that uses the patients bone conduction thresholds and type of hearing loss.

Everyday + DAI: Prescriptive Omni Plus fitting that uses the patients bone conduction thresholds and type of hearing loss on front microphone, DAI on rear microphone. Best for situations in which the patient wants to connect to DAI options while also hearing sounds around them.

DAI only: For use with FM systems and Bluetooth systems with no microphone input.

SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Main menu > Fitting > Initial Fit

Adjust the broadband gain, enable/disable the feedback canceller, enable/disable volume control, and change the input mode of the device. Right and left sides are displayed separately with response curves. Dotted lines on the response curves correspond to the target fit based on the patient's hearing loss.

1. Compare 2 alternate fittings.
2. Apply the target fit to the selected memory.
3. Mute/unmute
4. Undo
5. Serial number of connected device
6. Overall gain control
7. FB Cancellor: feedback canceller
VC Enable: enable volume control wheel
8. Select input mode.

Main menu > Fitting > Bands

Make adjustments to the gain of each frequency band.

1. Icons are described in Fitting > Initial Fit explanation (see above).
2. Adjust gain for each frequency band. Check the All slider to move all bands up or down.

SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Main menu > Fitting > Filters

Set notch filters to eliminate feedback. Two tabs display three notch filters each. Within each notch filter, set the frequency (Hz), the filter depth (dB), and the filter width (sharpness). The filter appears on the response graph as a divot in the curve.

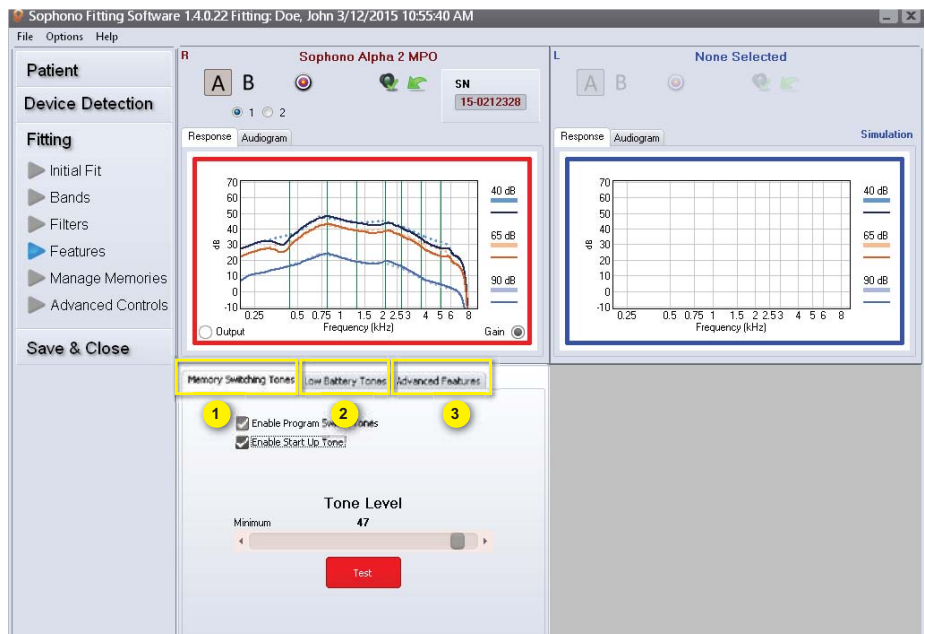
1. Two tabs of filters, each tab contains 3 notch filters
2. Within each filter –
Hz: Set frequency of the notch filter. dB: Set depth of the notch filter. Sharpness: Set the width of the notch filter.




Main menu > Fitting > Features

Enable/disable and adjust the volume of memory switching tones and low battery tones. Also adjust the AGCO and data logging in the Advanced Features tab.

1. Memory Switching Tones: Enable/disable memory. Test the sound using the Test button.
2. Low Battery tones: Enable/disable low battery tones. Test the sound using the Test button.
3. Advanced Features: Adjust AGCO
Adjust noise reduction
Enable/disable data logging

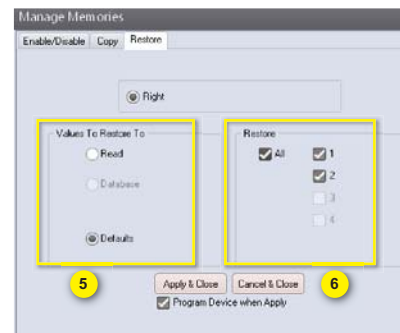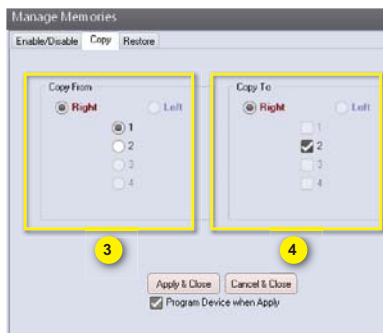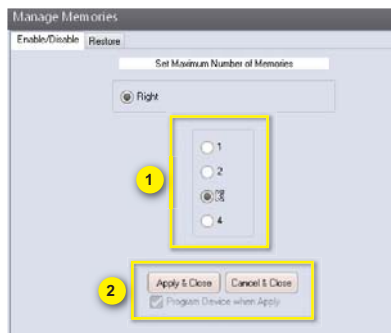SOPHONO™ SOFTWARE SCREEN FUNCTION DETAILS

Main menu > Fitting > Manage Memories

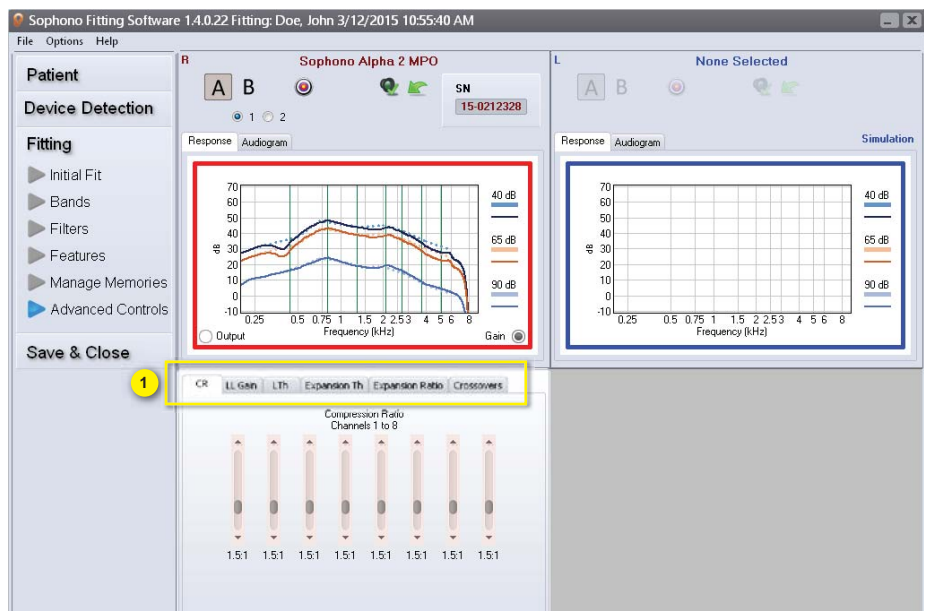
Set the number of memories, copy memories from one side to the other, or restore the memories to default or previous session settings.

1. Select the number of memories.
2. Select Apply & Close to apply changes.
3. Copy the memory from selection.
4. Copy the memory to selection.
5. Restore from selection
Read: settings stored on device when connected
Database: settings from stored session
Defaults: default setting
6. Restore to selection.

Main menu > Fitting > Advanced Controls

Change advanced parameters. Adjustments for these parameters are rarely needed. However, you may change the compression ratio, lower level gain, lower thresholds, expansion thresholds, expansion ratios, and crossovers.

1. CR (Compression Ratio): increases or decreases the amount of compression.
LL Gain (Lower Level Gain): increases or decreases gain for softer sounds (40 and 65 dB curves)
LTh (Lower Threshold): Increases or decreases the level at which compression kicks in.
Expansion Th (Expansion Threshold): Level below which expansion is active. Expansion is a reduction of the very soft sounds. Only applies if expansion ratio is turned on.
Expansion Ratio: Either off or on at 1:2. Expansion reduces gain for very soft sounds.
Crossovers: 8 channels in the device. Crossover is the frequency dividing the channels. The crossovers adjust the width of each of the channels.

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The screenshot shows the 'Device Detection' window of the Sophono software. It is divided into two main panels. The left panel, titled 'Sophono Alpha 2 MPO' and 'Sophono Alpha- 8 Channel - Bone Conduction Device', contains several settings: 'Use Connected Device - SN: 15-0106301' (selected), 'Simulate' (unselected), 'Programming Choice' with 'Quick Fit' (selected) and 'Manual Adjustment' (unselected), 'Patient Type' with 'Adult' (selected), 'Pediatric under 12' (unselected), and 'DAI' (unselected). Below these are 'Active Memories' (1, 2, 3, 4) and 'Autofit Options' (Sophono Ideal Fit, Noise Fit). The right panel, titled 'No Device Detected - Left', contains 'None' (selected) and 'Simulate' (unselected). At the bottom are 'Fitting' (6), 'Retry' (7), and 'Cancel' buttons. A red prohibition sign is over the 'No Device Detected' panel. Numbered callouts 1-7 point to specific elements: 1 points to the device name, 2 to 'Use Connected Device', 3 to 'Quick Fit', 4 to the 'Patient Type' section, 5 to the 'Autofit Options' dropdowns, 6 to the 'Fitting' button, and 7 to the 'Retry' button.

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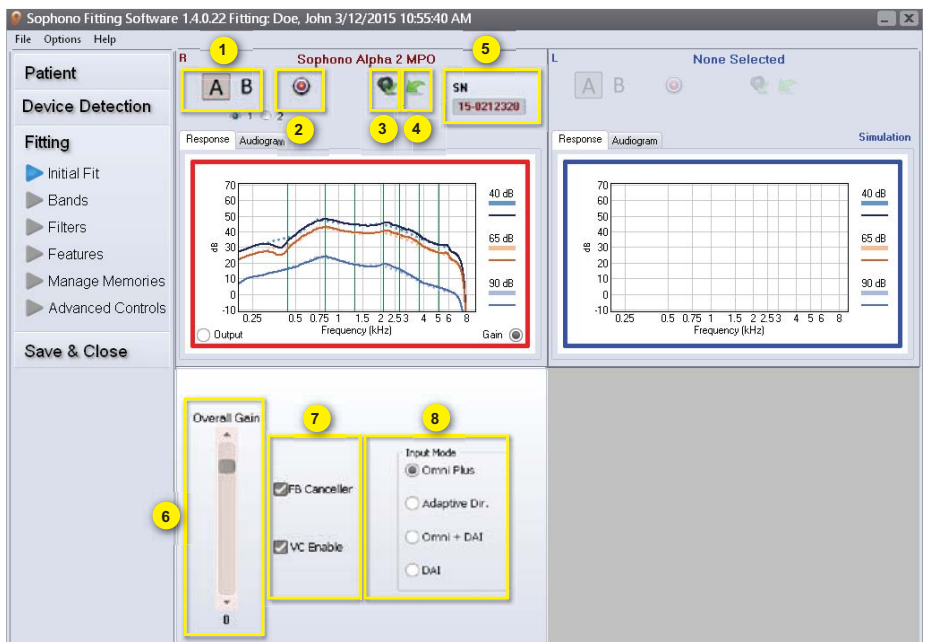
Menu principal > Fitting (Appareillage) > Initial Fit (Appareillage initial)

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Menu principal > Fitting (Appareillage) > Bands (Bandes)

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Menu principal > Fitting (Appareillage) > Filters (Filtres)

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Menu principal > Fitting (Appareillage) > Features (Fonctions)

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File Options Help

Patient

- ▶ Patient List
- ▶ Patient Info
- ▶ Counseling

Device Detection

Fitting

Save & Close

Last Name: Doe First Name: John Middle: Date of Birth: Primary Phone: Address: Email Address: Secondary Phone: City: ST/Province: Zip/Postal: Account Id: 10

Medtronic Further, Together

Right Device: SN: Selected Session Date: 2017-03-13 1:49:32 PM Left Device: SN:

dB HL vs Frequency (Hz) plots for Right and Left devices.

0k Hz: 55dB

0 10 20 30 40 50 60 70 80 90 100 110 120

125 250 500 1k 1.5k 2k 3k 4k 6k 8k

Frequency (Hz)

dB HL vs Frequency (Hz) plots for Right and Left devices.

0 10 20 30 40 50 60 70 80 90 100 110 120

125 250 500 1k 1.5k 2k 3k 4k 6k 8k

Frequency (Hz)

☐ Binaural ☒ Conductive Mismatch ☐ SSD

Edit ☐ AC ☒ BC

Clear Right Copy >> Delete Session << Copy Clear Left

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File Options Help

Patient

- ▶ Patient List
- ▶ Patient Info
- ▶ Counseling

Device Detection

Fitting

Save & Close

Sophono Product Information and Patient Counseling

Videos

- Introduction to the Sophono System
- Introduction to the Processor
- Peace of Mind

Processor

Model

- ☐ None Selected
- ☐ Brown
- ☐ Silver
- ☐ Champagne
- ☒ Anthracite

• Fully digital hearing system
• Dual-microphone Adaptive Directionality
• 8 Channels
• 4 Programs
• Direct Audio Input (DAI, Euro plug) port
• 128-band adaptive noise reduction
• Adaptive phase cancellation feedback management
• Log Data Logging
• Battery type: 13

dB HL vs Frequency (Hz) plot showing a shaded area representing the hearing range.

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Right Device: SN: Left Device: SN:

Selected Session Date: 2017-03-13 1:49:32

Fitting: ☐ Binaural ☒ Conductive Mixed ☐ SSD

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Videos

Introduction to the Sophono System Introduction to the Processor Peace of Mind

Processor

Model: ☐ None Selected ☐ Brown ☐ Silver ☐ Champagne ☒ Anthracite

Fully digital hearing system
Dual-microphone Adaptive Directionality
8 Channels
4 Programs
Direct Audio Input (DAI, Euro plug) port
128-band adaptive noise reduction
Adaptive phase cancellation feedback management
Log Data Logging
Battery type: 13

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The screenshot shows the 'Device Detection' window. On the left, under 'Sophono Alpha 2 MPO', it says 'Sophono Alpha- 8 Channel - Bone Conduction Device'. Below this are three sections: 'Use Connected Device - SN: 15-0106301' with radio buttons for 'Use Connected Device' (selected) and 'Simulate'; 'Programming Choice' with radio buttons for 'Quick Fit' (selected) and 'Manual Adjustment'; and 'Patient Type' with radio buttons for 'Adult' (selected), 'Pediatric under 12', and a checkbox for 'DAI'. Below 'Patient Type' is 'Active Memories' with radio buttons 1, 2 (selected), 3, and 4. To the right of 'Active Memories' is 'Autofit Options' with dropdown menus for 'Sophono Ideal Fit' and 'Noise Fit'. At the bottom left is an image of the device. On the right side of the window, it says 'No Device Detected - Left' and 'Make sure processor is turned on; check cables.' Below this are radio buttons for 'None' (selected) and 'Simulate'. At the bottom right is an image of the device with a red prohibition sign over it. At the very bottom are three buttons: 'Fitting' (labeled 6), 'Retry' (labeled 7), and 'Cancel'. Numbered callouts 1 through 7 point to specific elements: 1 points to the device name, 2 to the 'Use Connected Device' button, 3 to the 'Quick Fit' button, 4 to the 'Patient Type' section, 5 to the 'Autofit Options' dropdowns, 6 to the 'Fitting' button, and 7 to the 'Retry' button.

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© 2017 Medtronic, Inc.