## English

### Instructions for cleaning your BHM hearing system

In order to ensure the warranty and guarantee of your hearing system, please note that your hearing system must be cleaned thoroughly and properly at least once a day. Wipe your hearing device with a clean, dry cloth: Please make sure that your fingers are also clean and dry. This enables you to remove sweat and moisture from your hearing device.

Furthermore, it is necessary to remove the batteries from your hearing system at night after cleaning. Store overnight with the battery compartment open so that any moisture in the device can evaporate.

Remove your hearing device before showering, bathing or using the sauna, and before you put on body care products, hairspray or make-up powder. These deposits can soil the hearing device too, which may impair the flawless functioning of your hearing system.

We kindly request that you adhere strictly to these instructions, as unfortunately there is otherwise no guarantee claim i.e. guarantee repairs cannot be carried out.

BHM helps you keep your hearing system clean and dry. The BHM PerfectDry® drying box is an electronic drying system for hearing devices. It doesn't matter which BHM hearing system you use: simply place your cleaned device in the box before going to bed, switch it on, and that's it. After just two hours, your hearing system will be perfectly maintained and ready for trouble-free use.

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