pan

Digitales Hörmodul für Brillen
Digital hearing module for spectacles

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Key

Symbols	
	Manufacturer
	CE marking
	Do not dispose of in household waste
	Please read Instruction for Use
	Warning
	Temperature during transport and storage
	Humidity during transport and storage

Congratulations on acquiring your digital hearing module **pan** – a quality product from BHM. The digital hearing module for spectacle wearers is manufactured in Austria in line with the latest audiological findings and technical innovations and meets the requirements of 93/42/EEC.

pan is attached to glasses and worn with glasses.

By complying with the points listed in this Instruction for Use, **pan** is sure to be a reliable partner for many years to come.

The first few days of using a new hearing module will focus primarily on familiarising yourself with the module and becoming accustomed to the new sound impressions.

The amplification and functions of your hearing module have been tailored to your individual requirements by your hearing care professional.

How long it takes to become accustomed to the module may vary from person to person. Success often depends on a willingness to engage with the new sound impressions.

It is therefore advisable to try wearing your new hearing module regularly, even if it may feel a little strange to begin with.

Your hearing care professional will also instruct and inform you as regards first-time use, how to wear it properly and regular maintenance measures by **pan**.

You may of course contact your hearing care professional to discuss any questions or issues concerning your hearing or use of the hearing module.

Various individual settings, such as the sounding of warning tones or their volume is the responsibility of your hearing care professional. Please fit these settings together.

Intended Use: The hearing module **pan** has been designed to compensate hearing loss by processing sound and transmitting it to the ear drum.

User group(s): The hearing module is suitable for people of any age with hearing loss.

Indications: For compensation of hearing loss as well as for masking of tinnitus with **pan Ti**.

Contraindications: Do not use in case of irritated and injured skin or pressure marks.

Warning! Potential side effects: Possible pressure points or irritation of the skin.

Warning note for the hearing care professional:

This hearing module can generate an output sound pressure level of 132 dB SPL. Particular care must be taken when adjusting the module as there is a risk of damage to the user's residual hearing.

For additional, current information, please visit **www.bhm-tech.at**

Content

Before use	30
The hearing module pan	32
Getting started	33
Operation	36
Adjustment & programming	39
Tinnitus Masker	41
After use.....	43
Important information	44
Guarantee & repairs	47

Before use

Please read this Instruction for Use carefully before using your module. If you will not be wearing the hearing module yourself, please take note of the following information as relates to the person who will.

Please check that the functions and make sure that it is in proper condition. It must in no way be used if there is any suspicion that it does not work flawlessly or is damaged, especially the battery compartment.

Do not expose the hearing module to any excessive mechanical burdens. If it should fall on the floor or is otherwise damaged, please send it to your hearing care professional to be checked and repaired.

Never give the hearing module to others and/or prevent unauthorised use.

Components and batteries can be dangerous, cause severe injury and even be fatal if swallowed or handled in an improper manner.

Technical data	EN 60118-0 Ear simulator		EN 60118-7 / 2 cm ³ coupler		
	High Power Receiver	Normal Power Receiver	High Power Receiver	Normal Power Receiver	
Operating voltage	1,30 V		1,30 V		
Output sound pressure level (90 dB SPL input)	Max. 1600 Hz/HFA	130 dB SPL 123 dB SPL	127 dB SPL 119 dB SPL	118 dB SPL 114 dB SPL	115 dB SPL 111 dB SPL
Gain	Max. 1600 Hz/HFA	64 dB 56 dB	59 dB 51 dB	54 dB 49 dB	48 dB 43 dB
Frequency Range		100 Hz – 10000 Hz	100 Hz – 7500 Hz	100 Hz – 10000 Hz	100 Hz – 7500 Hz
Battery size	312		312		
Average battery life span (zinc air)	160 h		160 h		

The hearing module pan

- ① Program selection button
- ② Battery compartment
- ③ Programming connector
- ④ Receiver tube
- ⑤ External receiver
- ⑥ Earpiece
- ⑦ Microphones

Symbolic image 1

Getting started

Your hearing care professional will provide you with comprehensive instruction on how to handle your **pan** and accessories.

In order to guarantee the safety and functionality of the hearing module, only accessories from BHM must be used.

The external receiver


The receiver ⑤ and earpiece ⑥ selected to suit your requirements will be mounted on the **pan** by your professional hearing care professional. Your hearing care professional will instruct you fully on the handling, maintenance and daily care of your hearing module, external receiver and earpiece.

In the case of discomfort or skin irritation when wearing, please contact your hearing care professional directly.

 If parts of the hearing module come loose and get stuck in the ear canal, contact your hearing care professional immediately!

Battery compartment

Before using, open the battery compartment ② and make sure that you do not use a low battery (Symbolic image 2).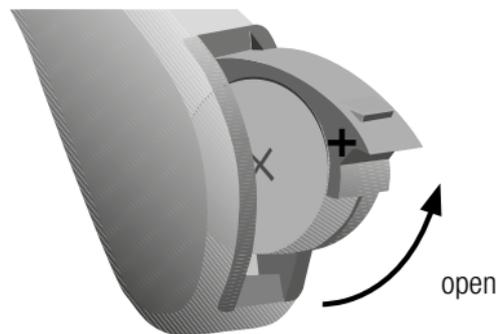

Symbolic image 2

Inserting or changing the battery

When inserting a battery, please make sure that the ⊕-pole is on the same side as the ⊕-sign

in the battery compartment. When replacing a battery, please ensure that you remove the old battery first before inserting the new one. Only use zinc air batteries in size 312. Remove the adhesive strip from the positive terminal. Then wait the amount of time stipulated by the manufacturer before inserting the battery into the hearing module.

Please note that a weak battery may impair the transmission characteristics of your hearing module.


pan does not have an ON/OFF switch. The module switches on when the battery compartment is closed and switches off when it is opened. After your day's use and/or when the module is not being used, we recommend storing it with the battery compartment open in a safe place out of reach of children. Opening the battery compartment also enables you to dry any moisture that has accumulated.

Applying the external receiver

Put on your spectacles, take hold of the receiver tube ④ at the bend and press the earpiece into the auditory canal (Symbolic image 3). The earpiece should be inserted into the ear until the receiver tube lies against it. Once the earpiece is correctly placed, the receiver tube should not be protruding from the ear when observed in the mirror.


Never try to change the shape of the tube system yourself.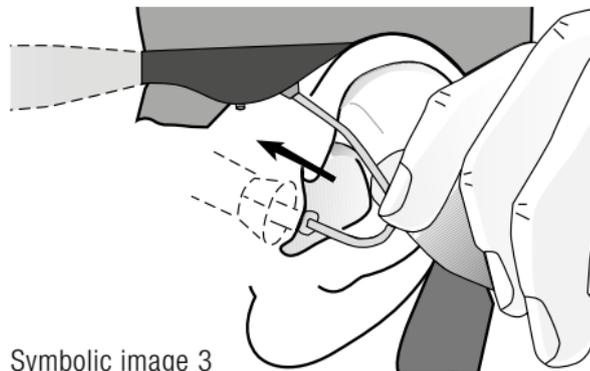

Symbolic image 3

Removing the external receiver

Take the receiver tube in your thumb and forefinger and withdraw it carefully from the auditory canal (Symbolic image 4). Please contact your hearing care professional if you have difficulty removing it.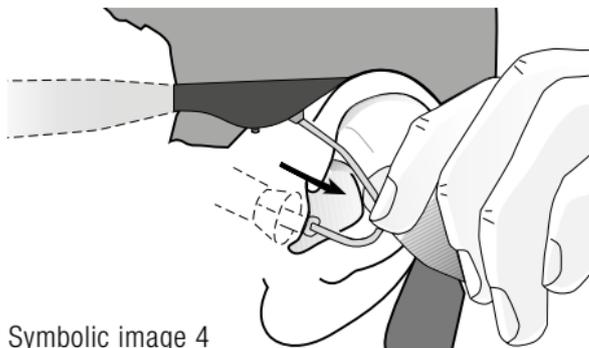

Symbolic image 4

Battery warning tone

Your hearing module will alert you when the battery voltage is low. A series of tones will remind you of this situation, until you change the battery or it runs out completely. The hearing

module will then switch off automatically. We therefore recommend always keeping a new battery readily available. All functions are automatically restored once a new battery has been inserted.

 Swallowing a battery is harmful to health and may cause severe damage or even death! Never change the batteries in the presence of infants, toddlers, children or mentally impaired persons. The insertion or replacement of the battery may only be carried out after instruction by the hearing care professional. Never store batteries alongside, for example, medicines! If you have swallowed a battery, please seek medical attention immediately. Always store batteries out of reach of children! Pay attention to an intact battery compartment!


Operation

Switch-on delay

When programming, your hearing care professional can activate a switch-on delay. This allows you to apply external receiver comfortably in the ear without any experiencing disturbing whistling caused by acoustic feedback. After closing the battery compartment, the hearing module will switch on after the programmed time.

Switching on/off

- If the battery compartment ② is fully closed, the hearing module will be switched on and programme 1 is activated.
- By opening the battery compartment ②, you will switch off the hearing module. The easiest way to do so is by opening the compartment with your fingernail (Symbolic image 5).

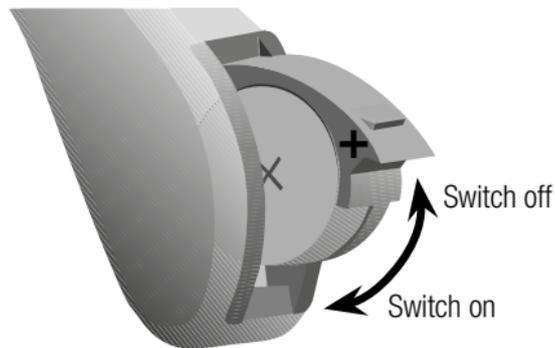Symbolic image 5

Programme selection button

If your hearing care professional has programmed several programmes into it, you can select these using the programme selection button. You will hear one or more tones that signal which programme you have just selected (1 signal tone = programme 1; 2 signal tones = programme 2; etc.). The programmes can be chosen by tapping the programme selection button (Symbolic image 6).

By switching off and on again, you will return by default to programme 1.

Symbolic image 6

For a better overview, please complete the following table (page 38).

Sound Dynamix (BHM Automatic)

In „Automatic“ mode, the signal received by the microphone is analysed and categorised into different hearing situations. These are:

- Quiet environments
- Speech in quiet environments
- Speech in background noise

- Music and movie
- Background noises
- Wind noises

The parameters are always configured according to this categorisation such that an optimal level of hearing comfort and optimal speech comprehension are attained.

Making calls

If your hearing module has been adjusted with a normal power receiver and an open tulip or dome earpiece, you can make calls quite normally by holding the telephone receiver to your auricle. If your hearing module has been adapted with a high power receiver with earpiece or individual otoplasty, it may be necessary to find an optimum position of the telephone.

The following tips should help you: Put the telephone receiver to your ear in the usual

Programme	Confirmation signal	Description of hearing situation
1	1 beep	
2	2 beeps	
3	3 beeps	
4	4 beeps	

way. Slide the telephone receiver near the microphone, but not directly on it (Symbolic image 7).

Symbolic image 7

If any acoustic feedback occurs, it may take a moment for the digital feedback suppresser to adapt. The acoustic feedback is also reduced when the distance between the telephone and the ear is somewhat bigger. According to your requirements, your hearing care professional can also set up a special telephone programme.

Adjustment & programming

(For hearing care professional only)

 **pan** is available as a monaural hearing module. For binaural support, two monaural modules are used. The decision regarding which module should be worn on the left-hand or right-hand side is made by the hearing care professional during the fitting.

Before programming

Before programming, please remove the socket cover (Symbolic image 8). Use a pair of tweezers to remove the socket cover. Under the cover, the programming socket to which the programming cable can be connected becomes visible (Symbolic image 9).

Insert the programming cable plug into the programming socket. Please make sure that the red marking on the programming cable is on the same side as the red marking on the programming socket. Otherwise, you will have no connection i.e. may destroy the programming socket! Only use the approved programming interface Hi-PRO.


Fitting software

In order to guarantee flawless function, please only use the BHM software released for this product. If you have any doubts or queries, please contact your supplier or BHM.

After programming

Remove the programming cable and replace the cover as follows:

- Grasp the socket cover using a pair of tweezers and place it back on the programming socket.
- Carefully press the cover back into the programming socket. Ensure that the cover is firmly seated on the programming socket (Symbolic image 10).

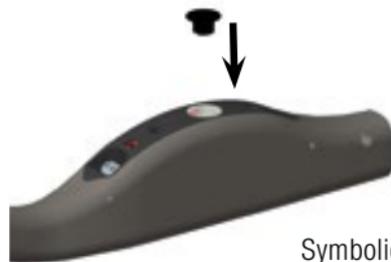Tinnitus Masker

(pan Ti only)

The Tinnitus Masker is a tool intended to generate noise to provide temporary relief for patients suffering from tinnitus as part of a tinnitus management program. The generated noise can be used as simple noise or be added to an input signal.

The Tinnitus Masker is targeted to hearing care professionals who are familiar with the evaluation and treatment of tinnitus and hearing loss.

Guidelines for users of the Tinnitus Masker

These instructions contain information about the Tinnitus Masker, which may have been enabled in your **pan Ti** hearing module by your hearing care professional.

The Tinnitus Masker is a tinnitus management tool intended to generate noise of sufficient intensity and bandwidth to help manage tinnitus. Your hearing care professional will also be able

to offer the appropriate follow-up care. It is important to follow your hearing care professionals advice and directions regarding such care.

Warnings related to tinnitus

- The Tinnitus Masker is a sound generator. It provides a means of sound enrichment therapy that can be used as part of a personalized tinnitus management program to provide temporary relief from tinnitus.
- The underlying principle of sound enrichment is to provide supplementary noise stimulation which can help defocus your attention from your tinnitus and avoid negative reactions. Sound enrichment, coupled with instructional counselling, is an established approach to managing tinnitus.
- Should you develop any side effects while using the Tinnitus Masker, such as headaches, nausea, dizziness or heart palpitations or experience a decrease in auditory function such as decreased loudness tolerance, worsening

of tinnitus, or speech not as clear, you should discontinue use of the module and seek a consultation with a medical, audiology, or other hearing care professional.

- The volume of the Tinnitus Masker can be set to a level that could lead to permanent hearing damage when used for a prolonged period of time. Should the Tinnitus Masker be set to such a level in your hearing aid, your hearing care professional will advise you of the maximum amount of time per day you should use the Tinnitus Masker. The Tinnitus Masker should never be used at uncomfortable levels.
- There are some potential concerns associated with the use of any sound generated by a tinnitus management module. Among them are the potential for worsening of tinnitus and/or a possible change in hearing thresholds.

Important information related to tinnitus

- The Tinnitus Masker generates sounds that are used as part of your personalized temporary tinnitus management program to provide relief from tinnitus. It should always be used as prescribed by a hearing care professional who is familiar with the diagnosis and treatment of tinnitus.
- Good health practice requires that a person reporting tinnitus have a medical evaluation by a licensed ear physician before using a sound generator. The purpose of such an evaluation is to ensure that medically treatable conditions, which may be causing tinnitus, are identified and treated prior to using a sound generator.
- Always follow the instructions of your hearing care professional regarding the wearing time per day. Do not exceed the maximal wearing time per day. Prolonged use may lead to worsening of your tinnitus or of your hearing loss.

After use

Maintenance and cleaning

- Please be advised by your hearing care professional as regards the maintenance and daily cleaning of your hearing module.
- Influences such as dirt, wind, cold, heat, moisture, leaky batteries, daily use, ageing module, etc. may have negative effects on the properties of the module. In order to minimise the likelihood of a malfunction, the module must be cleaned and maintained regularly.
- Pollution, sweat and moisture should be removed from your hearing module daily (wiped with a soft, dry cloth).
- If not using, it is also necessary to remove the battery every evening after its daily clean. When not in use, please store it with the battery compartment open in order for any moisture present in the module to evaporate.

- The receiver and earpiece must be cleaned regularly. Under no circumstances use water to clean it, but only a soft, dry cloth.
- Please take off your hearing module before showering, washing your hair, stepping into a sauna or steam room.
- Remove the hearing module whilst applying perfume, creams, hairspray or other cosmetics.
- Keep your hearing module dry and clean.
- Never try to dry the hearing module with a hair-dryer or in a microwave oven.
- For particularly hygienic and protective cleaning, we recommend special products available from specialist stores. Under no circumstances use petrol or similar chemical cleaning products.

We kindly request that you follow these instructions at all times, as otherwise there can be no guarantee claim and no warranty repairs can be carried out!

Disposing of batteries

Empty batteries must be disposed of responsibly within the meaning of environmental protection. As such, please take these items to a special disposal site. Your hearing care professional will be happy to help you in this.

Disposing of the hearing module

Your hearing module is a modern technology product that must be disposed of responsibly within the meaning of environmental protection. As such, please take these items to a special disposal site. Your hearing care professional will be happy to help you.

Important information

The basic settings of your hearing module are put in place by authorised experts according to the particular hearing loss at hand. The user must therefore only use the easily accessible operator elements intended for him or her, as per instructions.

Improper use of your hearing module and battery may endanger your health. Please read the following points with great care.

- Keep other people away from your **pan** hearing module and its components, when it is not in use! There is the risk of swallowing small parts!
- Adjust the volume according to individual requirements and upon instruction by your hearing care professional.

- **pan** is able to emit signals at a high volume. In order to prevent possible hearing damage, the hearing module must be adapted by a qualified hearing care professional.
 - **Swallowing batteries puts your health at risk and can cause serious injury and even death! Never change the batteries of the hearing module in the presence of children and mentally impaired persons. The insertion or replacement of the battery may only be carried out after instruction by the hearing care professional. Please seek immediate medical attention in the event of swallowing a battery.**
 - **Incorrect handling and use of incorrect batteries may cause the battery to leak and endanger your health! Always use size 312 hearing aid batteries and replace them regularly.**
- **Improper handling of the hearing module may loosen its smallest components. Please do not swallow small detachable components, this may harm your health and can have serious consequences!**
 - Never expose your hearing module to extreme solar irradiation (for example on your car windshield), extreme heat (do not place on heating modules) or extreme cold (overnight in a cold car) and avoid contact with chemicals or harsh detergents.
 - The penetration of liquids into the module can lead to malfunctions and consequently to health problems.
 - Please always be aware that hearing modules will no longer function properly if, for example, battery power is running out. This poses a particular danger in traffic or when you are in any other way dependent on warning signals.

- Sweat and moisture should be removed from your hearing module on a daily basis (wipe with a dry cloth). Please remove your hearing module before showering, washing your hair, stepping into a sauna or steam bath.
 - For a particularly deep clean or a mild clean that is gentle on the skin, we recommend special products available from specialist stores. Under no circumstances use petrol or similar chemical cleaning products.
 - A faulty hearing module must be repaired by a specialist. Do not try to open the module yourself. Improper handling will result in the guarantee becoming void.
 - If your hearing module is defective, please consult your hearing care professional. Regular maintenance and checking can prevent repairs from becoming necessary.
 - In the case of interference fields with electromagnetic interference such as transmitters, the hearing quality of your hearing module may be impaired.
- The hearing module must be removed before undergoing any special diagnostic tests such as MR scans, x-rays, radiotherapy, etc.
 - For safety reasons (risk of swallowing and suffocation), children or people with a mental impairment may only wear the hearing module under supervision.

Do not use your hearing aid in places where electronic aids are prohibited.

Only connect devices that comply with the relevant international safety standards.

- In the case of a malfunction or defect in your **pan** module, please contact your hearing care professional immediately. Do not try to open or repair the hearing module yourself. Improper handling will result in the guarantee and warranty becoming void. Regular maintenance and checking with your hearing care professional can prevent repairs.

Guarantee & repairs

BHM guarantees for material and manufacturing defects in the hearing module as stated in the guarantee conditions. In its service concept, BHM guarantees safe function through the use of BHM original components or hearing modules. There is a one-year guarantee period for the hearing module; the warranty is two years. Your hearing module requires regular maintenance. Contact your hearing care professional for this.

If your BHM hearing module does not work, it must be repaired by a qualified service technician. Do not try to open the hearing module casing, as this will void the guarantee and warranty for the hearing module. The guarantee does not cover any defect or damage to your hearing module resulting from the use of wearing options that do not come from BHM.

PROBLEM	POSSIBLE CAUSES	POSSIBLE SOLUTIONS
Hearing module doesn't work	<ul style="list-style-type: none"> • Hearing module not switched on • Empty or weak battery • Battery compartment doesn't close • Blocked earpiece • Blocked cerumen filter 	<ul style="list-style-type: none"> • Switch on hearing module (close battery compartment) • Replace battery with a new one • Insert the battery correctly • Clean the earpiece • Replace the cerumen filter
Hearing module is too quiet	<ul style="list-style-type: none"> • Earpiece has been placed too loosely • Blocked earpiece • Hearing loss has changed • Too much cerumen (ear wax) in the auditory canal • Volume is set too low • Blocked cerumen filter 	<ul style="list-style-type: none"> • Remove the earpiece and reinsert • Clean the earpiece • Consult your hearing care professional • Consult your hearing care professional • Consult your hearing care professional • Replace the cerumen filter

PROBLEM	POSSIBLE CAUSES	POSSIBLE SOLUTIONS
Hearing module whistles	<ul style="list-style-type: none">• Earpiece has been placed too loosely• Dome earpiece is not correctly placed• The settings are not optimal• Dome earpiece is worn or defective	<ul style="list-style-type: none">• Remove the earpiece and reinsert it• Reinsert the dome earpiece• Consult your hearing care professional• Consult your hearing care professional
Sound is distorted	<ul style="list-style-type: none">• Weak battery• Earpiece not placed correctly• The hearing module is damaged• The settings are not optimal• Volume is too low	<ul style="list-style-type: none">• Replace the battery with a new one• Consult your hearing care professional• Consult your hearing care professional• Consult your hearing care professional• Consult your hearing care professional

For all other problems, please contact your hearing care professional directly.

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