## Feedback

- Be sure that no hair is caught between the processor and the baseplate.
- Be sure that no hair is covering the microphones.
- Try moving the processor around a bit on the baseplate. Sometimes slight adjustments will remove the feedback.
- Be sure that both magnets in the baseplate are aligned with both magnets in the implant.
- Enable the Feedback Canceller using the Fitting Software.
- Use notch filters in the Fitting Software to eliminate feedback frequencies.
  - Use the Feedback Detector built into the software (found under Help) OR a phone application to detect the frequency.
  - Go to the filters window of the fitting software and set a notch filter for that frequency.
- If feedback persists at a specific frequency, decrease the gain in the bands tab of that frequency range by a few dB.

## Intermittent

- Check that the battery is fresh.
- Check that the battery door is fully closed.
- Contact Sophono Technical Support (1-877-604-9441 ext. 2).

## Sound Quality

Patient Comment	Solution
It sounds like I am in a tunnel or barrel, it sounds boomy	Reduce the low frequency gain. Go to the bands window and decrease the 125 Hz and 500 Hz bands by a few dB.
It sounds very echo-y	Reduce the low frequency gain. Go to the bands window and decrease the 125 Hz, 500 Hz, and 1000 Hz bands by a few dB.
The sound is tinny/sharp	Reduce the high frequency gain. Go to the bands window and decrease the highest 3 or 4 bands by a few dB.
I hear a whistle or humming sound	This is feedback – follow steps above for feedback control
It sounds distorted	Reset the device to default settings and start over. Distortion typically occurs if too many settings have been altered on the device.
I hear sound but cannot make out the words (muffled)	Increase the high frequency gain. Go to the bands window and increase the high frequency bands by a few dB.

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