SOPHONO™ INSTALLATION GUIDE

Using installation CDs

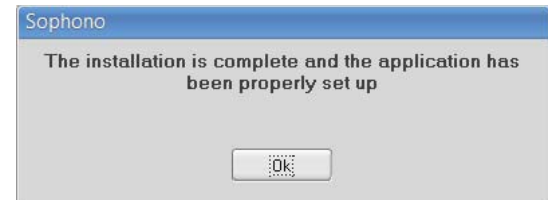
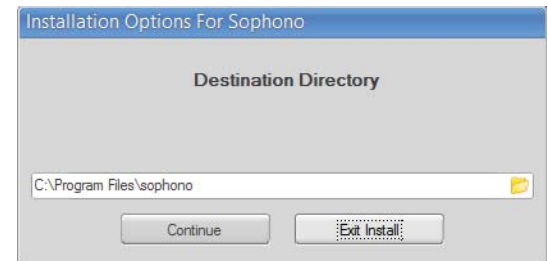
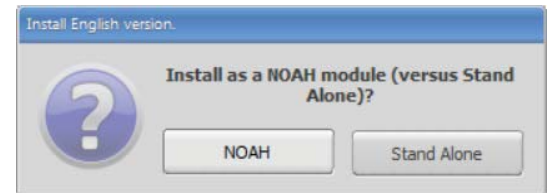
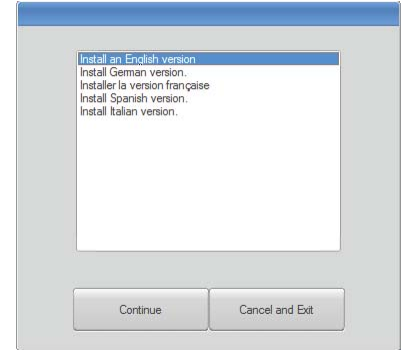
Prior to installing the Fitting Software, if the programmer software is not installed, perform the following steps, otherwise skip to Step 2.

Step 1: Install the programmer (HiPro or NOAHlink) software and configuration

- Insert the HiPro Installation CD into the disk drive
 - Follow the automated steps of the CD
 - Run the HiPro configuration and ensure the HiPro is on COM Port 1-4. If it is not, see the HiPro Troubleshooting tips.
- Insert the NOAHlink Installation CD into the disk drive
 - Follow the automated steps of the CD
 - Run the NOAHlink Properties program and ensure the computer can connect to the NOAHlink. If it cannot, see the NOAHlink Troubleshooting tips.

Step 2: Install the Sophono software

- Insert the Sophono software CD into the disk drive
- Run the .exe file and downloading will begin
- When prompted, select the installation language
- Select either NOAH or stand-alone for the install type
- Then select the destination directory. The automatic folder is the ideal location.
- If the software is successfully installed, you will see a confirmation message.

SOPHONO™ INSTALLATION GUIDE

Using the Sophono Webpage

Step 1: Navigate to the download page on the Sophono Website

- Go to Sophono.com
- Navigate to the Software downloads (For Professionals... Fitting Software)
- Complete the required fields. The password will be sent via email.
- If not already installed, install the programmer (HiPro or NOAHlink) software first.
- Install the Fitting Software. Installation steps will look the same as downloading the software with the CD.

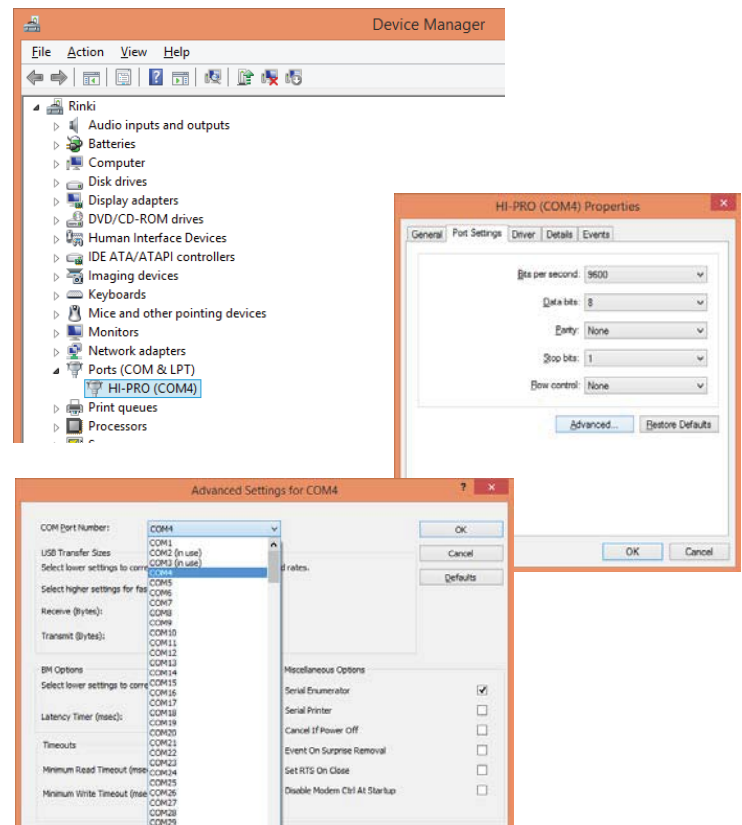
Step 2: Configure the programmer

- For HiPro
 - Run the HiPro Configuration
 - Ensure the HiPro is on COM Port 1-4. If it is not, see the HiPro Troubleshooting.
- For NOAHlink
 - Run the NOAHlink Properties program
 - Ensure the computer can connect to the NOAHlink. If it cannot, see the NOAHlink Troubleshooting tips.

HiPro Troubleshooting Tips



- When the HiPro is connected to the computer, a solid green light is on. If the HiPro does not show this, contact GN Otometrics.
- Ensure the HiPro is on COM Port 1-4.
 - Run the HiPro Configuration.
 - If the COM Port is listed as anything other than 1-4, you will need to manually change it.
 - Go to the device manager on your computer
 - Under Ports, find the HiPro
 - Right click on the HiPro and select properties
 - Go to Port Settings...Advanced...
 - Change the COM Port to 1-4
- Re-run the HiPro Configuration

SOPHONO™ INSTALLATION GUIDE

NOAHlink Troubleshooting Tips

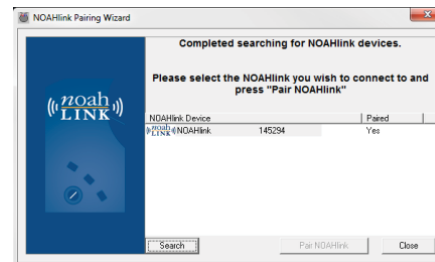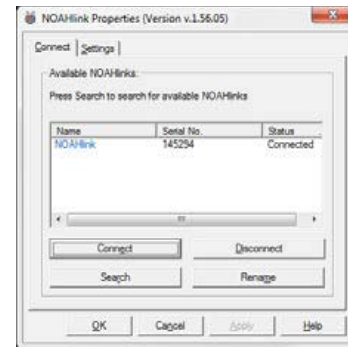- When the NOAHlink is powered on, a flashing green light is indicated. If the NOAHlink does not show this, try replacing or recharging the battery.
- Ensure the computer can connect to the NOAHlink.
 - Run the NOAHlink Properties program
 - Press the Search button
 - Select the name of the desired NOAHlink device
 - Press the Connect button
 - Once the connection is established, close the NOAHlink Properties program.
- If the NOAHlink still does not appear in the list.
 - Run the NOAHlink Pairing Wizard
 - Press the Search button
 - Select the name of the desired NOAHlink device
 - Press the Pair NOAHlink button
 - Once the pairing is established, close the program
 - Re-run the NOAHlink Properties program as detailed above

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© 2017 Medtronic, Inc.